---

# User and maintenance instruction manual

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
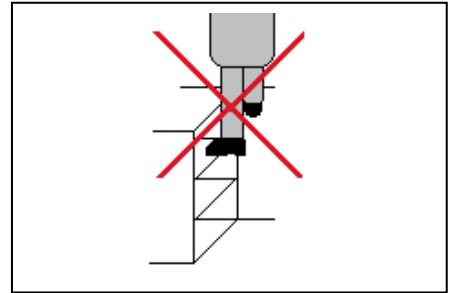
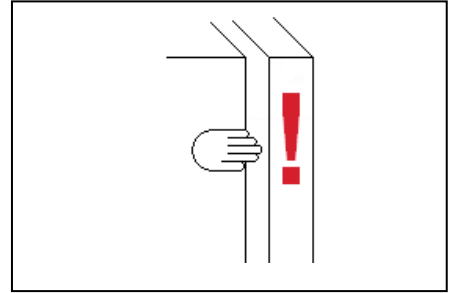
Do not keep flammable substances near appliance.

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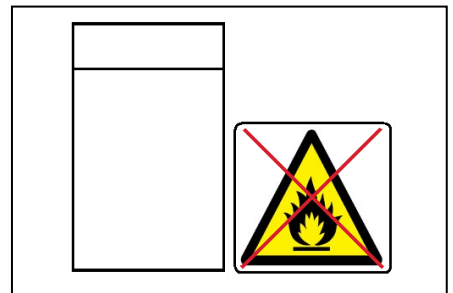Servicing must be carried out by qualified maintenance technician.

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**Caution!** Appliance contains refrigerant and electrolytic capacitor.

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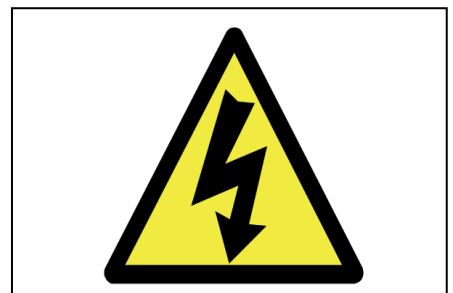
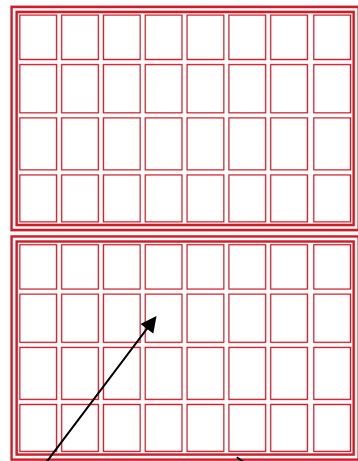
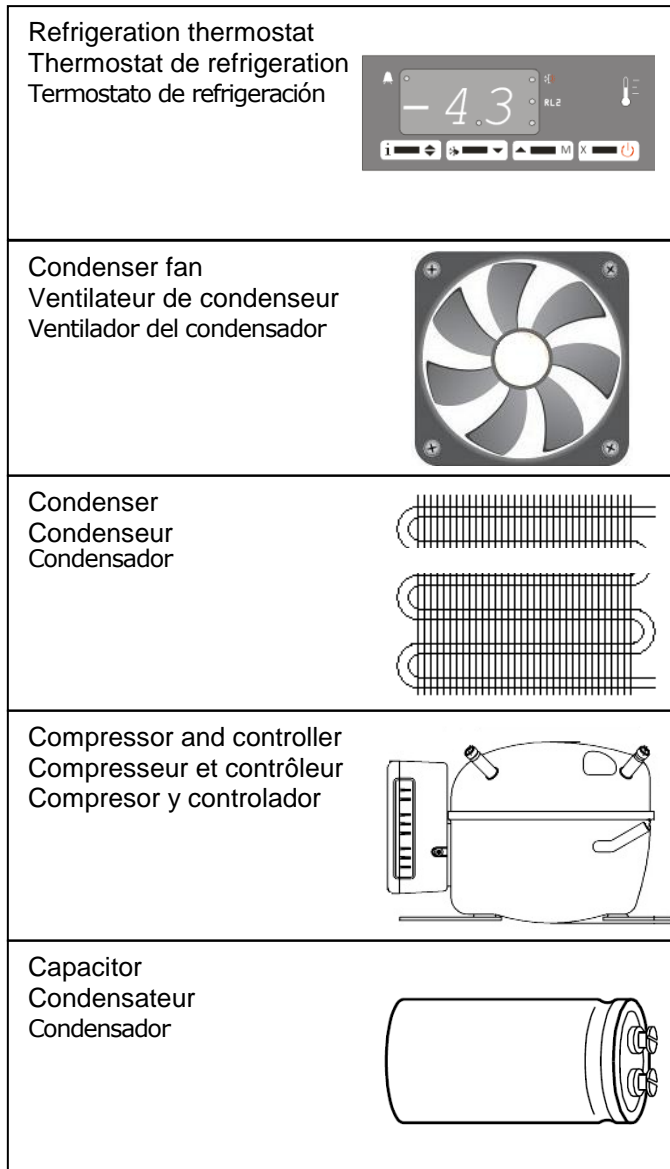
**Warning!** Appliance is solar powered all parts are live in daylight.

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
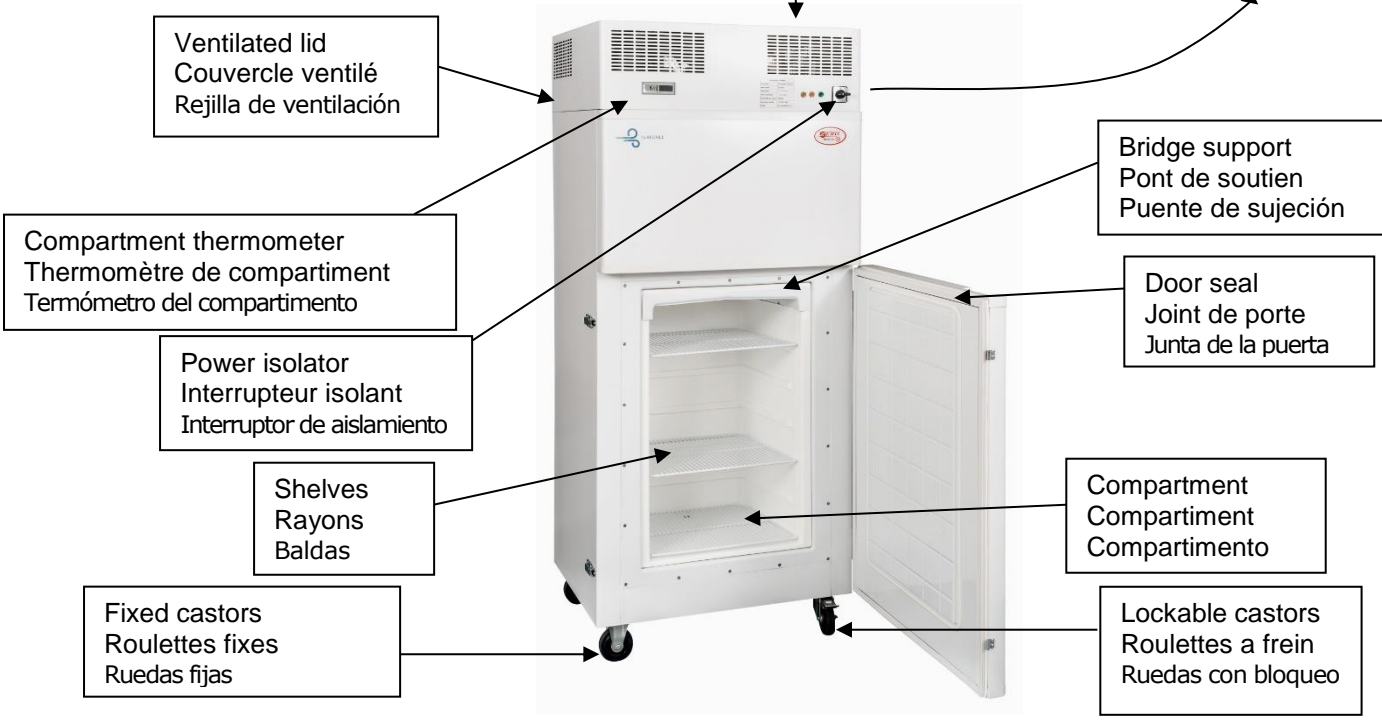
Compartment temperature is displayed on appliance in °C.  
Thermometer is solar powered and requires light to operate correctly.

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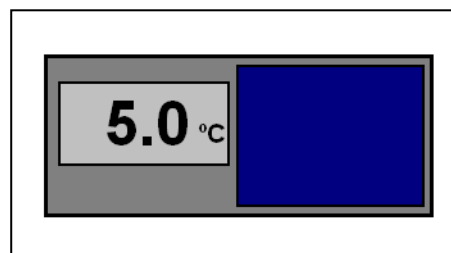It is normal for appliance to be cooled to 4-6°C within 6-8 days after commissioning\*.

The appliance is thermostatically controlled. Due to the unique design and temperature controls, vaccines will never freeze.

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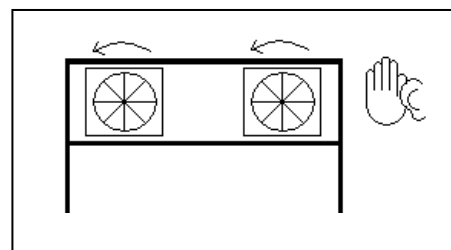
Fans operate and are audible when the compressors are working.

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
In high humidity condensation will form on the inside of the compartment.

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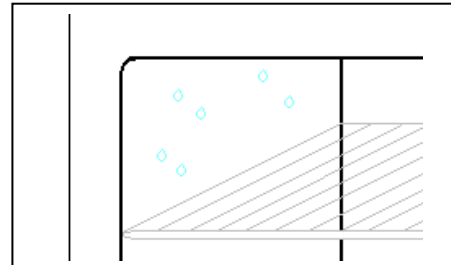
Wipe all compartment surfaces and door seals with clean, dry cloth.

Remove

And empty the external drip tray.

Do not use abrasive materials to clean the vaccine refrigerator.

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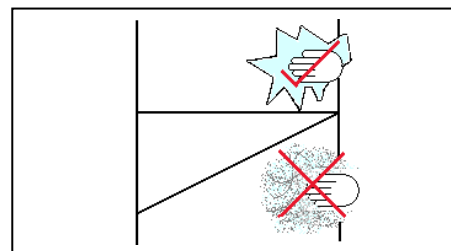
This appliance does not require defrosting.

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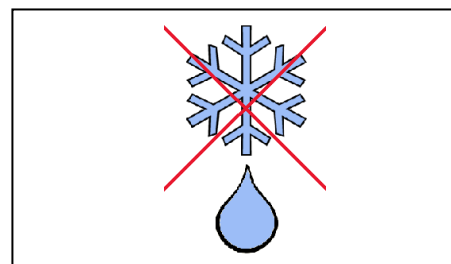
When appliance has been deactivated appliance can be transported and stored.

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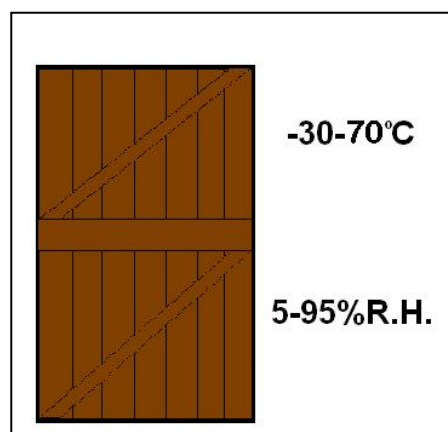There is three indicator lights installed indicating the following:

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1 – When the left amber light is lit it indicates that the left cooling system is functioning correctly.

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2 – When the right amber light is lit it indicated thr right cooling system is functioning correctly.

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3 – When the green light is lit it indicates there is power to the appliance and the isolator switch is in the ON position.

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NB: DURING LOW SOLAR RADIATION CONDITIONS THE THREE LIGHTS WILL SWITCH ON AND OFF INTERMITTENTLY. AS SOON AS SOLAR RADIATION CONDITIONS IMPROVE THE LIGHTS WILL FUNCTION NORMALLY.

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IF SOLAR RADIATION LEVELS ARE HIGH AND ONE OF THE TWO AMBER LIGHT STAY OFF THIS COULD BE A COMPONENT FAILURE IN THE COOLING SYSTEM. WHEN THIS OCCURS PLEASE CONTACT A QUALIFIED SERVICE TECHNICIAN.

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## Maintenance tasks/ Entretien/ Mantenimiento

**Daily** - confirm compartment temperature is within acceptable range and check all vents are unobstructed.

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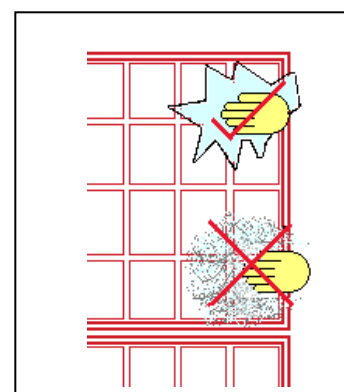
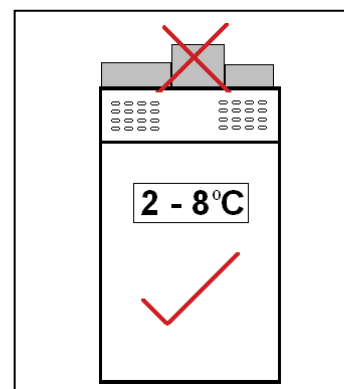
**Monthly** – check solar modules are unshaded and wipe with a clean cloth. This task should always be performed in the morning when the solar module is cold. Do not use any detergent on the solar modules.

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**Please advise your maintenance technician immediately if appliance is not functioning correctly.**

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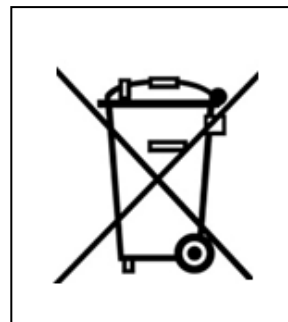
This appliance must be disposed of in accordance with EU WEEE Regulations.

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
Danger! Risk of child entrapment – **Remove door before disposal.**

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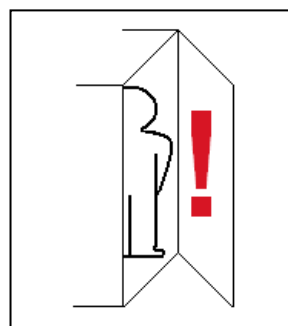**Refrigerante:** R600a — recuperar y reutilizar

**Outer casing:** Zinc Plated Mild Steel

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**Foam:** Polyurethane Foam

**Mousse:** Mousse de polyurethane

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**Plastic:** Various – Contact manufacturer for advice

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