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### Weekly

- Wipe clean the inside of the refrigerator.
- Wipe clean refrigerator door seals.
- Check for build up of condensation – clean water trap if necessary.

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### Defrosting / dégivrage

This unit will **not require defrosting** under normal working conditions – The internal faces of the refrigerator compartment are the cooling element and none should go below 2 degrees at any time during the operation. Therefore frost cannot form on the internal faces of the compartment.

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### Transportation / Transport

The unit can be transported and stored (when the product is inactivated) in ambient temperatures ranging from -30°C to +55°C and can be transported, stored and used (activated) in relative humidity from 5% to 95%.

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## Handling

- Units are supplied to enable handling most easily by forklift truck or pallet truck. You are advised by the World Health Organisation to limit the lift by one individual to 25kg or less.
- Keep the unit vertical.
- Do not tip the unit further than 30 degrees.
- Keep the door and latches closed when moving the refrigerator.
- Allow 24 hours after transportation before starting the compressor.

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## Positioning and cleaning

- Position the unit with free air all around – keep all faces at least 100mm (4”) away from walls.
- Position the unit in the shade and ensure it is NEVER in direct sunlight.
- In order to remove the top cover, to fill and undertake maintenance work it is necessary to have a clearance of at least 300mm from the top of the unit. Ideally there should be 600mm+ clearance above the top of the unit.

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Ensure the unit is level when in position and before filling or starting the unit.

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- Remove screw cap and add the 50ml of blue liquid provided as part of phase change material process.

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- Fill the container with exactly 150L of drinkable water.

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- Check level of water with dipstick provided.

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- If water level is low, add more water until required level is reached.

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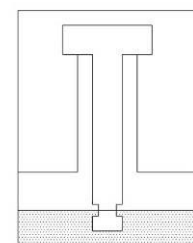
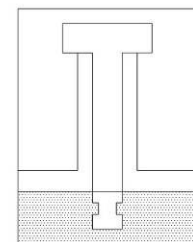
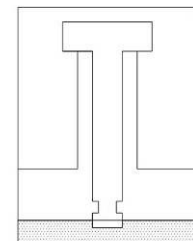
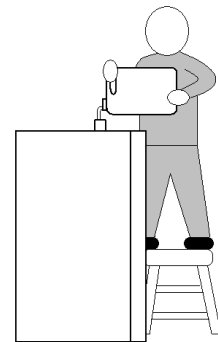
- If water level is too high, remove some water.

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- The correct water level is reached when the water level is between Indicated marks on the dipstick.

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Observe safe working practices when filling the appliance

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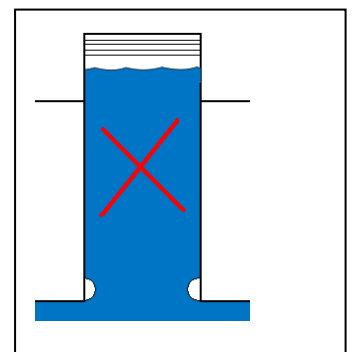
- Allow 24 hours after transportation before starting the compressor.
- Plug in the unit and switch on – There are no user adjustable parameters on this unit.
- The unit is fitted with an Automatic Stabilised Protector (ASP). This unit may cause some delay before starting so allow some minutes to pass after first connecting the unit.
- **No external voltage stabiliser is needed as this refrigerator is fitted with an Automatic Stabilised Protector unit (ASP).**

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- If the temperature indication on the refrigerator compartment goes below 4°C then call a maintenance engineer to look at the thermostat control.
- If the temperature is above 7°C then:
- Check the door is closed and sealing properly – If seals are damaged then report this and get the seals changed.
- Check that the thermometer is reading correctly – The thermometer sensor is on the inside back face. Use something of known temperature to assess the calibration of the thermometer. Also check the compartment temperature with a portable thermometer e.g. glass/alcohol type.
- Check the mains supply.
- Check if the unit has been receiving electricity within the allowed tolerance of voltage in the recent past.
- Check connection to the mains.
- If mains ok check that the green light on the Automatic Voltage Protector (ASP) is lit. If not, call an electrical engineer to look at the Automatic Voltage Protector (ASP).
- If 3,4,5 & 6 all appear to be in order then remove the vaccines to safe location and, ensuring constant supply of electricity, monitor the refrigerators progress over a 1 week period.
- When the compressor is running the fan should also operate – It should be possible to hear the running of the fan and feel air blowing out of the front vent at the top of the refrigerator.

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– When the left amber light is lit it indicates that the cooling system is functioning correctly.

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3 – When the green light is lit it indicates there is power to the appliance.

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NB: IF THERE IS POWER TO THE APPLIANCES AND THE GREEN LIGHT IS LIT BUT THE AMBER LIGHT IS OFF THIS COULD BE A COMPONENT FAILURE IN THE COOLING SYSTEM. WHEN THIS OCCURS PLEASE CONTACT A QUALIFIED SERVICE TECHNICIAN

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<b>SPECIFICATIONS</b>	<b>REFRIGERATOR</b>	<b>FREEZER</b>
Vaccine Storage Capacity	150 litres	N/A
Manufacturer's gross volume	161 litres	
External dimensions (h x w x d) - Without casters	1865 x 850 x 710 mm	

<b>ELECTRICAL</b>		
Voltage	145V - 290V AC	
	Fitted with Automatic voltage Stabilised Protector	
Spike Protection	160J, 6500 Amps (8/20s) Response time <10ns	
Frequency	50Hz	
Temperature range	-10°C to +55°C ambient	

<b>REFRIGERATION CIRCUIT</b>		
Refrigerant	R134a	
Refrigerant Charge	230 g	
Capillary Length	2600 mm	
Capillary Diameter	0.042 mm	

<b>SPARE PART S</b>	
<b>Part</b>	<b>Part number</b>
Compressor	IMPC20
Solar Thermometer	IMP60
Condenser	IMP65
Fan	IMP70
Temperature Controller	IMP75

## Refrigeration Circuit Product Disposal WEEE Regulations

- This symbol means that the product is classed as Electrical or Electronic Equipment and is covered by the European Union WEEE Regulations.
- Que Ce symbole signifie que le produit est classé comme équipement électrique ou électronique et est couvert par le Règlement de l'Union européenne WEEE.
- You should take great care in disposing of any such products and **MUST NOT** under any circumstances dispose of it with your normal waste as they pose a great risk to your community if they are accidentally sent for disposal with landfill.

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### Customer support

#### Contact company:

Zero Appliances  
Mission Terrace,  
Chloorkop Ext. 44,  
Kempton Park  
Johannesburg  
South Africa

Tel: +27 10 207 1600

Fax: +27 10 207 1675

Website: [www.zeroappliances.co.za](http://www.zeroappliances.co.za)